
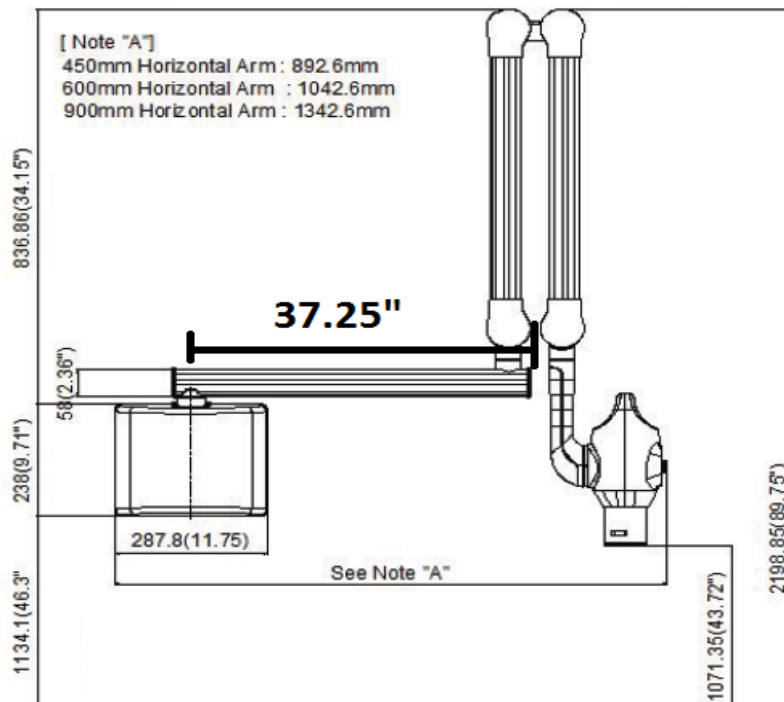
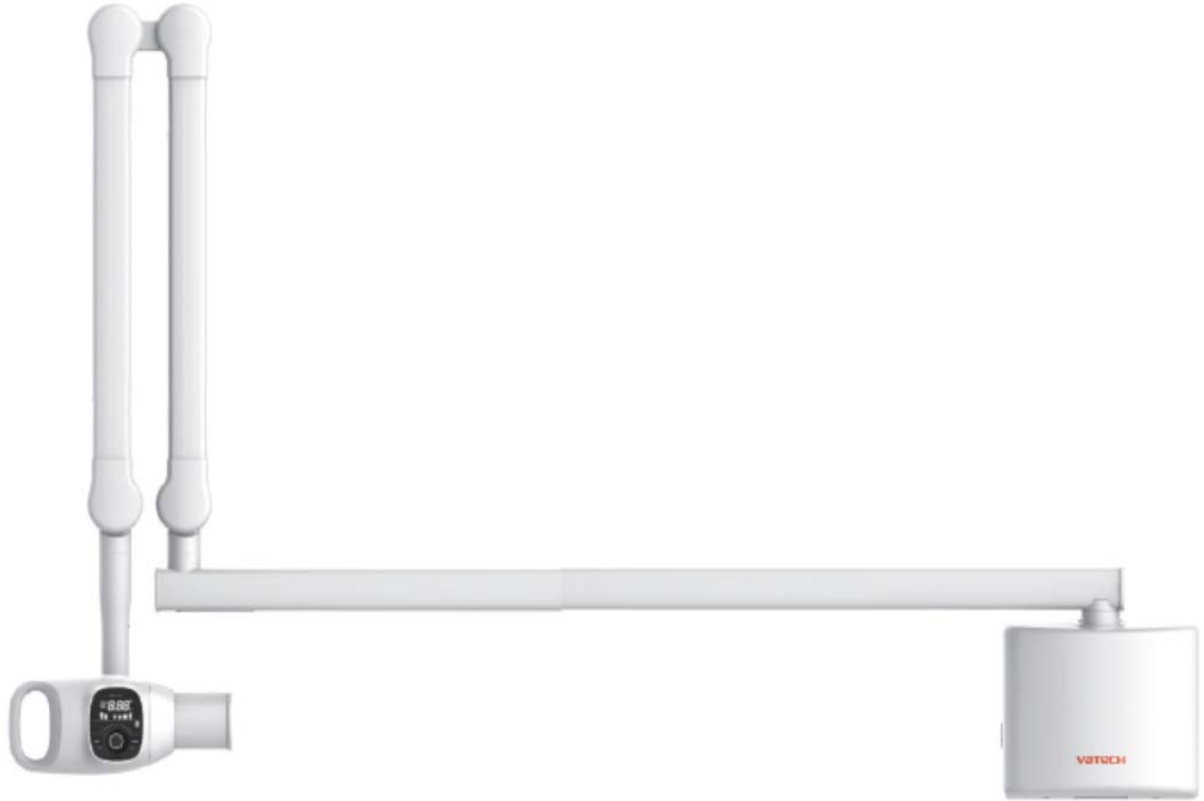
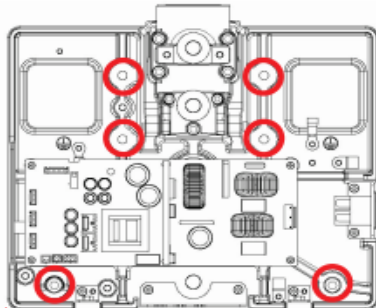
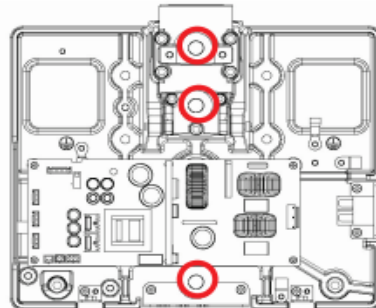


EzRay Air w

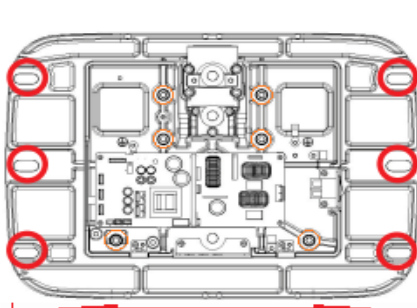


FOR ADDITIONAL INFORMATION, PLEASE REFERENCE THE INSTALLATION MANUAL
<https://vatechamerica.exavault.com/share/view/lmlq-1h0stkgy>

WALL MOUNTING & DIMENSIONS


Cabinet Mount Installation	Single Stud Installation	Double Stud Installation
 <p>Wood wall 11.75" - Wood screw X 6 (Part No.13) Concrete wall - Anchor X 6 (Part No.19) - Anchor wrench bolt X 6 (Part No.18)</p>	 <p>Wood wall - Wood screw X 3 (Part No.13) Concrete wall - Anchor X 3 (No.19) - Anchor wrench bolt X 3 (No. 18)</p>	 <p>Wood wall 17 10/16" - Wood screw X 6 (Part No.13) Concrete wall - Anchor X 6 (Part No.19) - Anchor wrench bolt X 6 (Part No.18)</p>

CONNECTING CABLES


Connect the cable H001624 with the Connector 1, 2 on the Power Box Board, and the Protective Earth (PE) Cable on the PE Conductor.

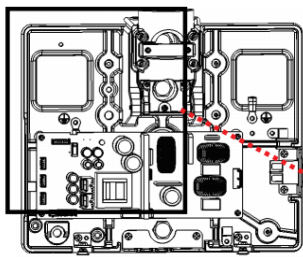

* Cable names base on models lengths
 *1. 450mm H001624
 *2. 600mm H001625
 *3. 900mm H001626
 Installation is the same on all lengths

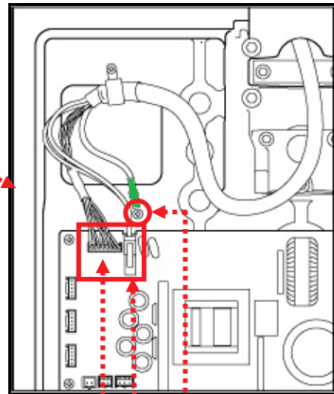

CN (Connector) 2 Protective Earth (PE) Cable
 CN (Connector) 1

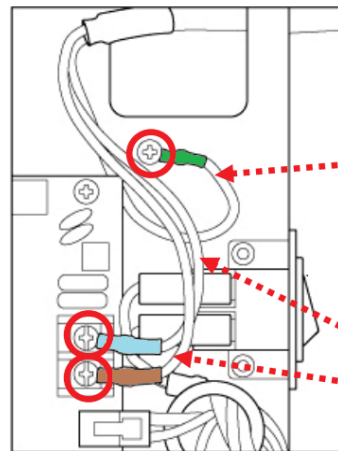

Protective Earth (PE) Cable

N (Neutral) (Light Blue)

L (Live) (Brown)

ELECTRICAL SPECIFICATION REQUIREMENTS

Power Supply Voltage	100-120v
Frequency	50/60 Hz (Single phase)
Power Rating	Max 350VA
Tube Voltage	65kV fixed (+-3%)
Tube current	Option A 2.5mA; Option B 3.0 mA
Accuracy Error	kVp +10% mA +20%

EzRay Air W TOTAL WEIGHT

900mm including power box assembly and X-ray generator assembly

15.45 kg / 34lbs

600mm including power box assembly and X-ray generator assembly

14.60 kg / 32 1/4lbs

450mm including power box assembly and X-ray generator assembly

14.15 kg / 31. 1/4lbs