



## **Ez3D Installation Guide for Vender and IT Consultants**

1. There are two types of installation; it is important to understand those two main types

A. "Console" – is the Capture

B. "Workstation" – Any remote viewing station; which is not the Capture pc.

2. The installation files should be either located on the CD provided with the Dental Xray Device; on the capture pc or network share (if already copied or downloaded) or it can be downloaded from Vatech America Dealer share FTP or by contacting a Support representative to share the link for download.

3. Run the Ez3D Plus installation file.

NOTE: (if this is an upgrade to existing installation it will update continue on with next until it finishes step 6 , if it says it can't be upgraded you will need to remove Ez3D Plus (or Ez3D 2009 from add remove programs and then rerun the installation wizard.)

4. During setup read each step carefully you will hit next a couple of times entering information and agree to end user terms; once you arrive at the "Select type" section you will do as follows depending on the type of installation you are performing.

A. Console – For capture pc only

B. For workstation- all systems other than capture pc.

5. Clinic next the rest of the way through.

6. Click 'finish' once it is completed.