

Green CT 2 - 3D TMJ/AIRWAY Patient Instruction Guide

To Patient:

Please follow these instructions to capture a 3D TMJ/Airway Scan:

Patient Positioning

- 1) **Hands** – Palms facing up on the bottom of the handle bar
- 2) **Feet** – Heels aligned with column, feet shoulder width apart
- 3) **Head** – Chin on chinrest, and align teeth in the grooves of the bite stick

Patient Instructions

- 1) Swallow and tongue remains at rest in your mouth
- 2) Bite down on back teeth
- 3) Breathe through your nose
- 4) Close your eyes
- 5) Remain still for 30 seconds



VATECH
ASSURANCE