

Green CT 2 – 2D Pan & 3D Patient Instruction Guide

To Patient:

Please follow these instructions to capture an Xray image:

Patient Positioning

- 1) **Hands** – Palms facing up on the bottom of the handlebar
- 2) **Feet** – Heels aligned with column, feet shoulder width apart
- 3) **Head** – Chin on chinrest, and align teeth in the grooves of the bite stick

Patient Instructions

- 1) 2D - Swallow and position ENTIRE tongue to the roof of your mouth
3D - Swallow and keep tongue at rest in your mouth
- 2) Breathe through your nose
- 3) Close your eyes
- 4) Remain still for 30 seconds



VATECH
ASSURANCE