

Green CT 2 – 3D (CBCT) Patient Instruction Guide

To Patient:

Please follow these instructions to capture a 3D (CBCT) Scan:

Patient Positioning

- 1) **Hands** – Palms facing up on the bottom of the handlebar
- 2) **Feet** – Heels aligned with column, feet shoulder width apart
- 3) **Head** – Chin on chinrest, and align teeth in the grooves of the bite stick

Patient Instructions

- 1) Swallow and keep tongue at rest in your mouth
- 2) Breathe through your nose
- 3) Close your eyes
- 4) Remain still for 30 seconds



VATECH
ASSURANCE