


## STEP 1. Prepare the Device for Packing

Remark

Check

- ① In Capture software, press Ctrl + S to bring up Serial Communication window

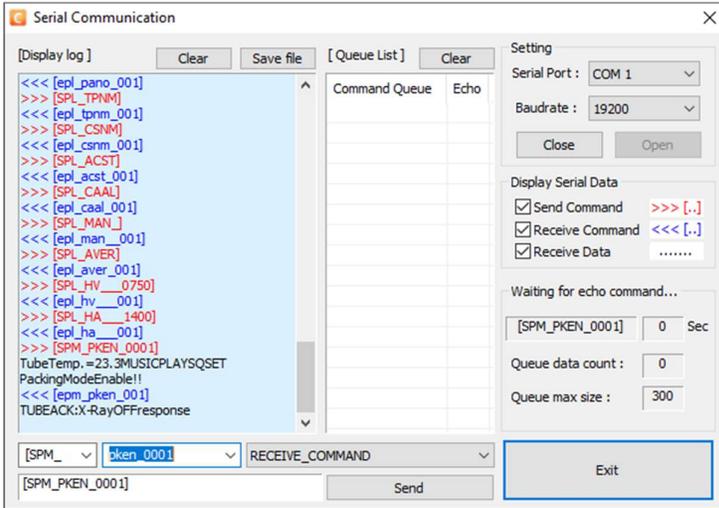

Please mark 'X' in box after you complete each step.




- ② Enter 'pken\_0001' in upper right box and Click 'SEND' to put the unit in Packing Mode
- Once you hear the chime and see [epm\_pken\_001], the device is moving into packing mode
  - Restart the unit to confirm the unit does not initialize

← A Unit moves into locking position once you click 'SEND'



- ③ Install the 2 M8x45 locking bolts as shown below to secure the rotator from moving

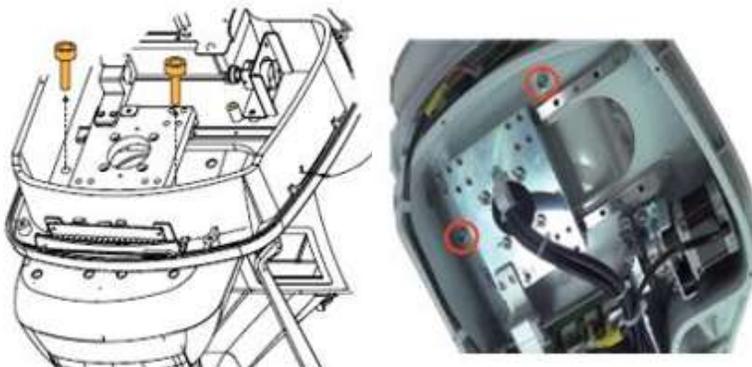


← Locking Bolts are included in the Empty Box


← This is to prevent the Rotator from moving during transportation.

← **If not locked, the device will be mechanically damaged.**

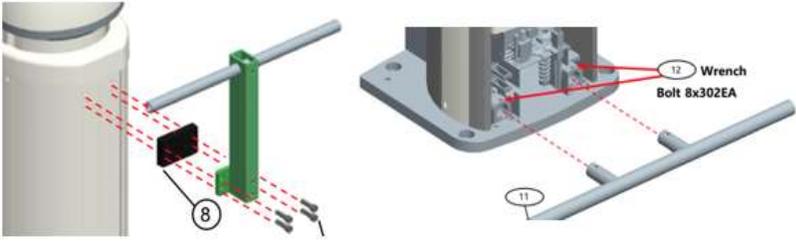

<p>④ Install provided packing bars (Upper and Lower)</p> <ul style="list-style-type: none"> <li>○ Upper packing bar requires Spacer (#8) and 4 M8x35 bolts</li> <li>○ Lower packing bar requires 2 M8x30 bolts</li> <li>○ Do not cross thread or overtorque the bolts</li> <li>○ Make sure the rotator is locked.</li> </ul> 	<p>← Moving Bars and Bolts are included in an Empty Box</p>	<p>Please mark 'X' in box after you complete each step.</p> <div style="text-align: center; border: 1px solid black; width: 40px; height: 40px; margin: 20px auto;"></div>
--	---	--

**STEP 2. Place the device in the Packing Box for Return**
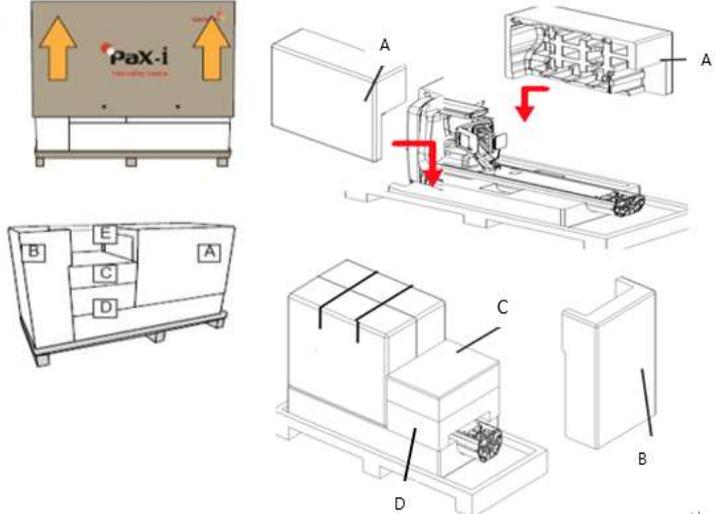
<p>① Boxing instructions</p> 	<p>← Be sure to properly box up the unit.          ← Take note of placement of each foam and how they're applied          ← <u>Any missing, cutting Box or broken foam will be billed</u>          ← <u>Packing Bars must return with the unit otherwise it will be billed.</u></p>	<div style="text-align: center; border: 1px solid black; width: 40px; height: 40px; margin: 20px auto;"></div>
--	---	--

I confirm that the device is properly packed following by the Packing Guide.

Please place the paper back in the Packing Box after signing.

Signature : \_\_\_\_\_

Print Name : \_\_\_\_\_

Date : \_\_\_\_\_