



Greyfinch – IOX and EOX Integration


1. First, open the desired patient record using the search box in the top left.
2. Then, type their name in the field on the popup.
3. Finally, click on the correct record that is displayed in the results:

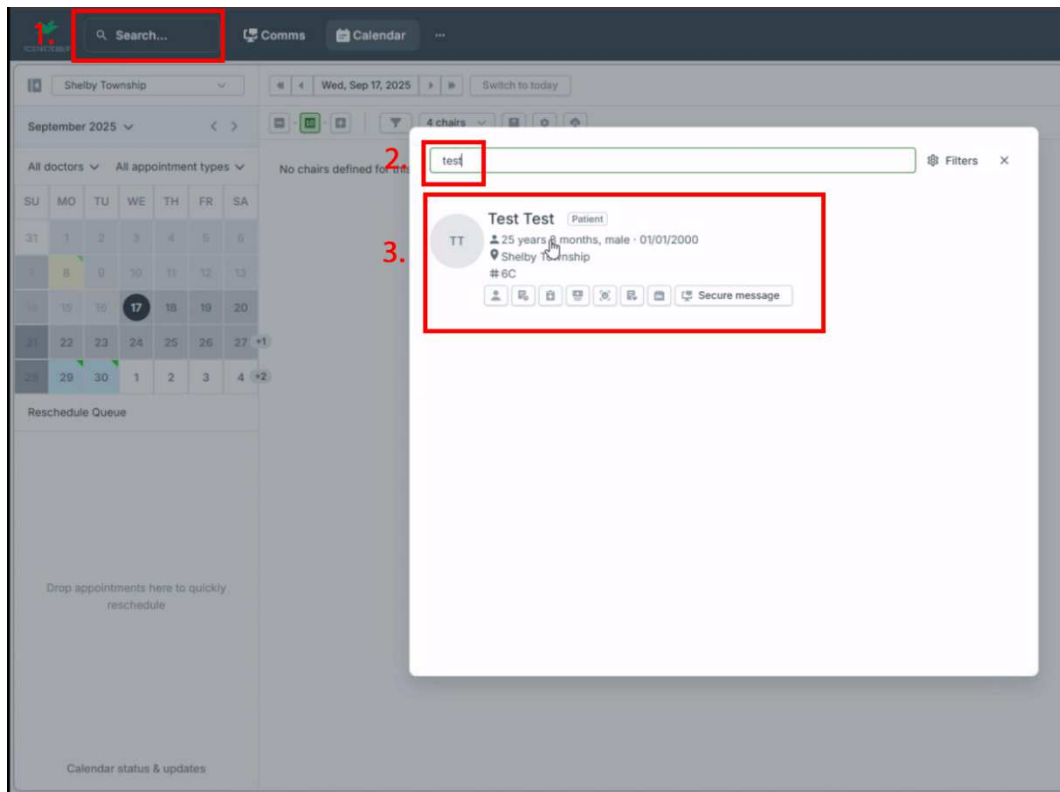


4. With a patient open, click on the icon with the “smiley face framed in four right-angle corners” to navigate to their imaging module:

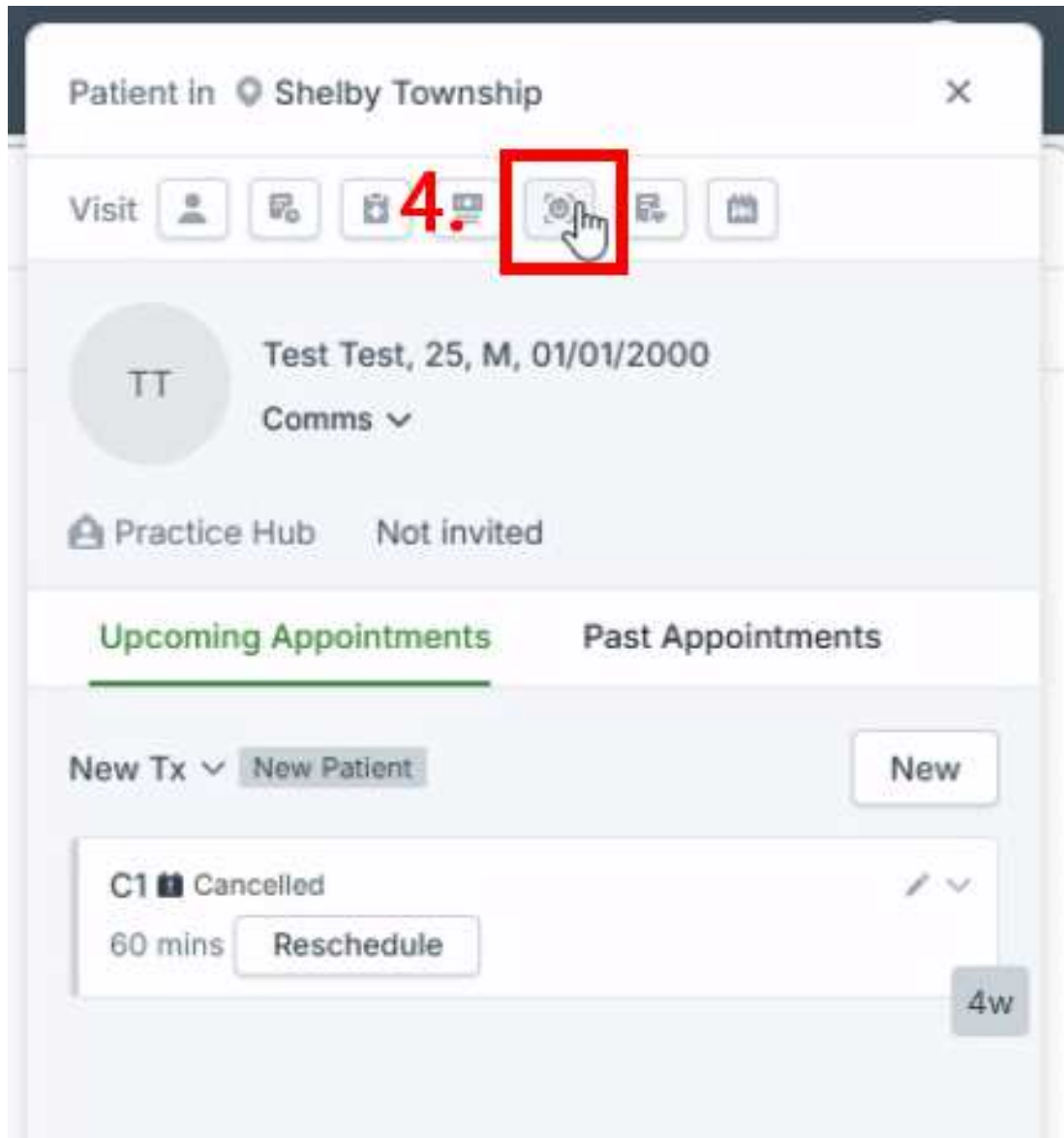


5. Next, in a “New image series,” click on the “Import” dropdown.
6. Then, click on “Import from device” from the list of options:

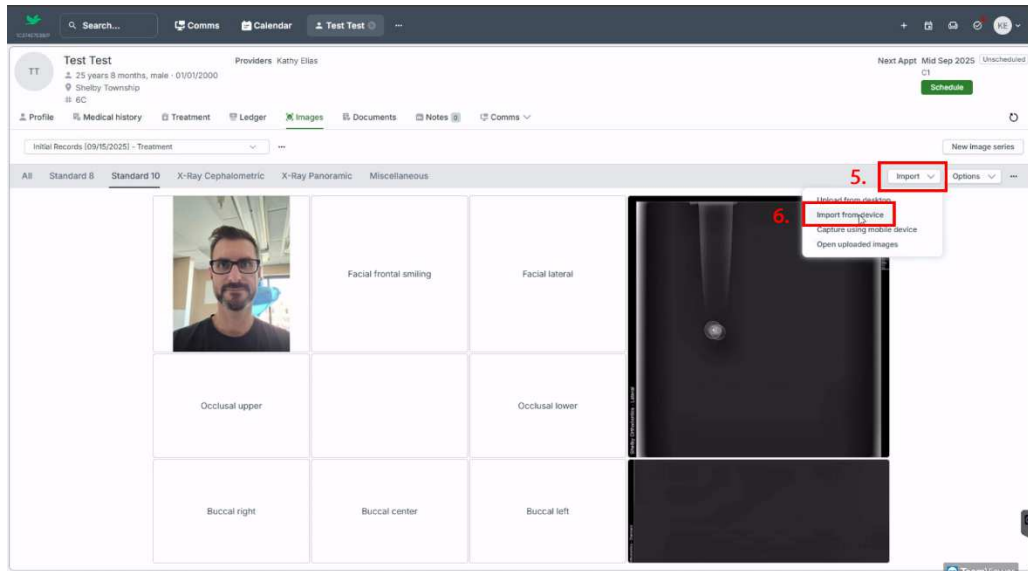


7. If this is the first time setting up any TWAIN devices on the computer this message will be displayed. Click “Download” to continue:

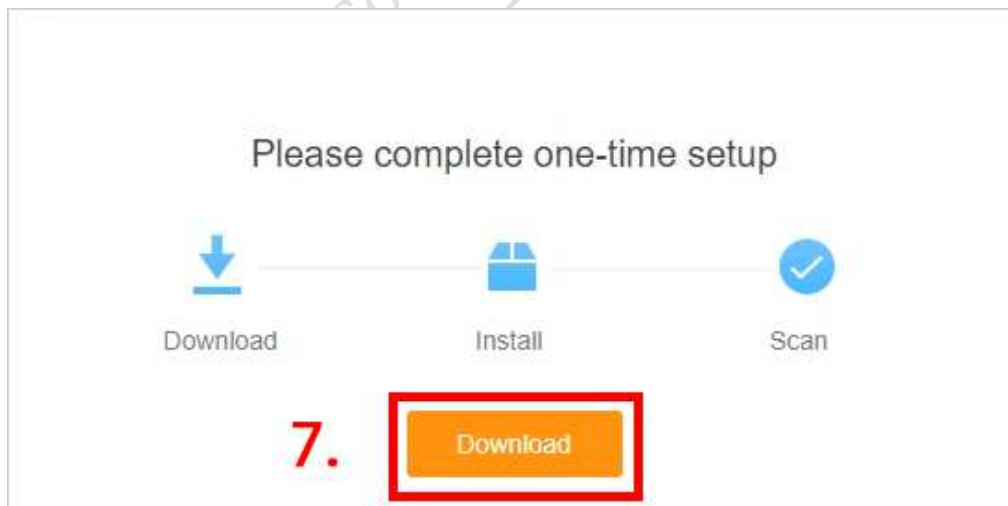


8. Reveal the downloads list in the browser by clicking on the down arrow pointing to a tray button.
9. Click on the DynamsoftServiceSetup.msi file that was recently downloaded to run the installer:

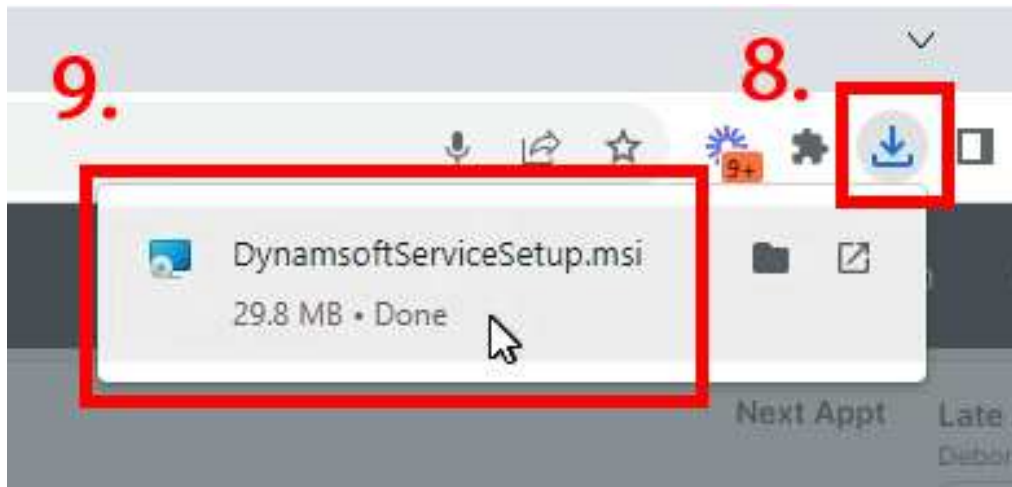


10. On the Dynamsoft Service Setup Wizard, click “Next” to proceed:

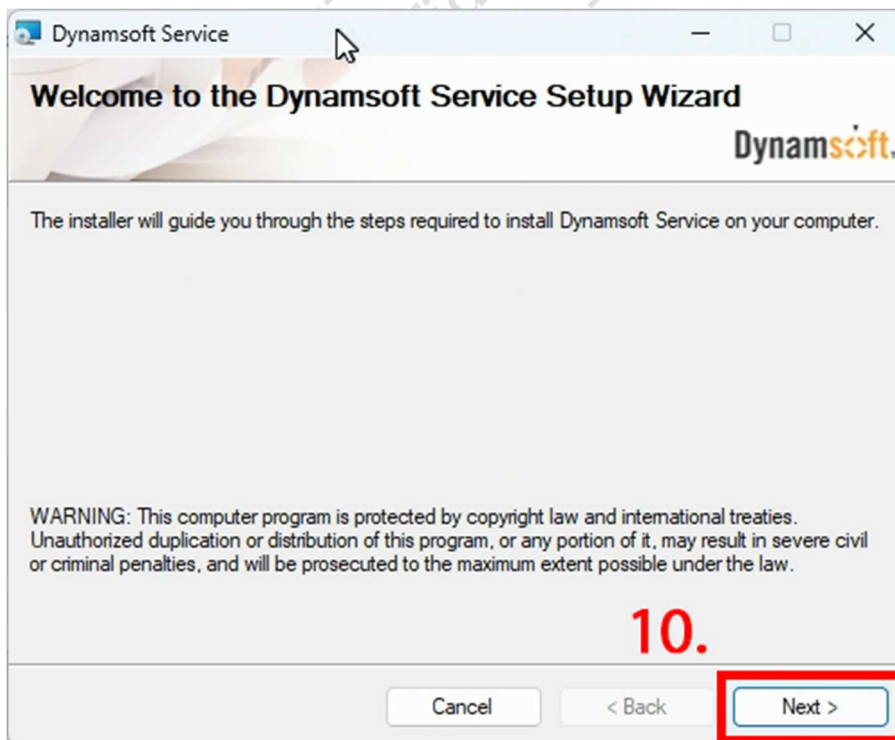


11. Click “Next” again to start the installation:

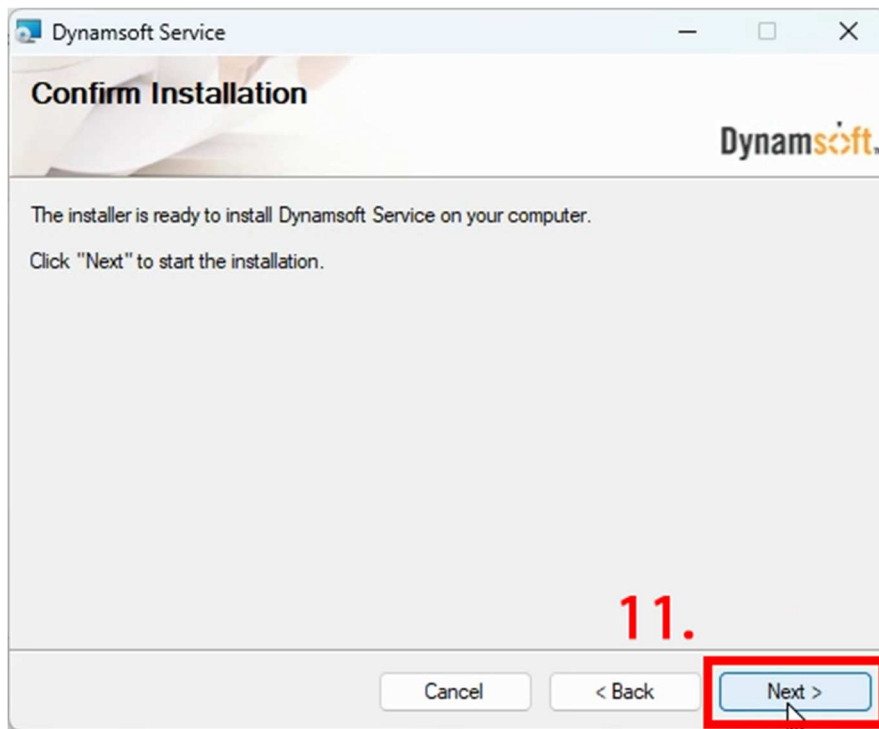


12. Wait for the progress bar to fill up and for the installation to be completed:

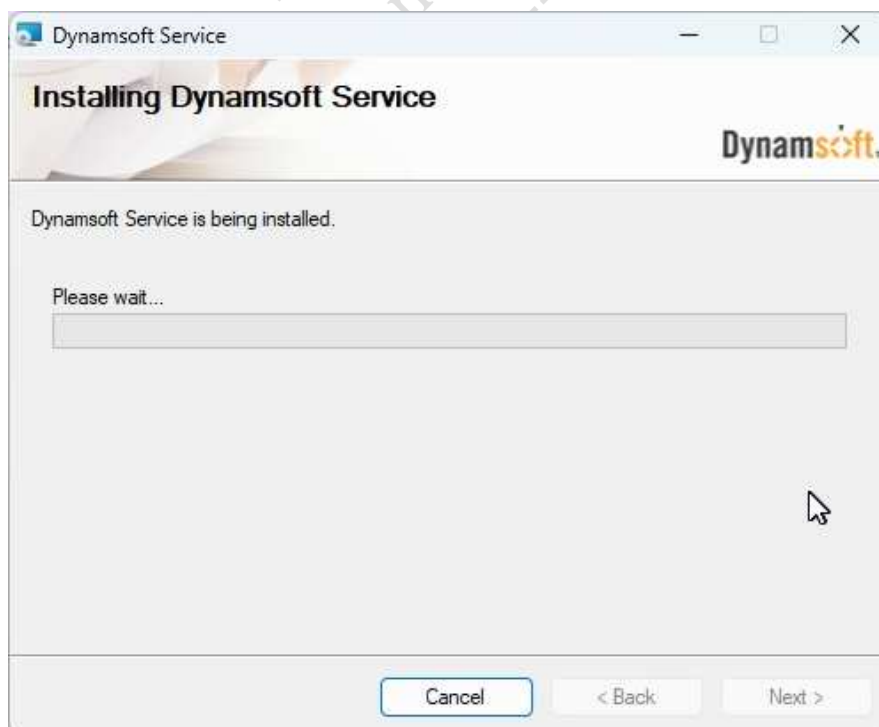

13. Finally, click “Close” to return to Greyfinch in the web browser:

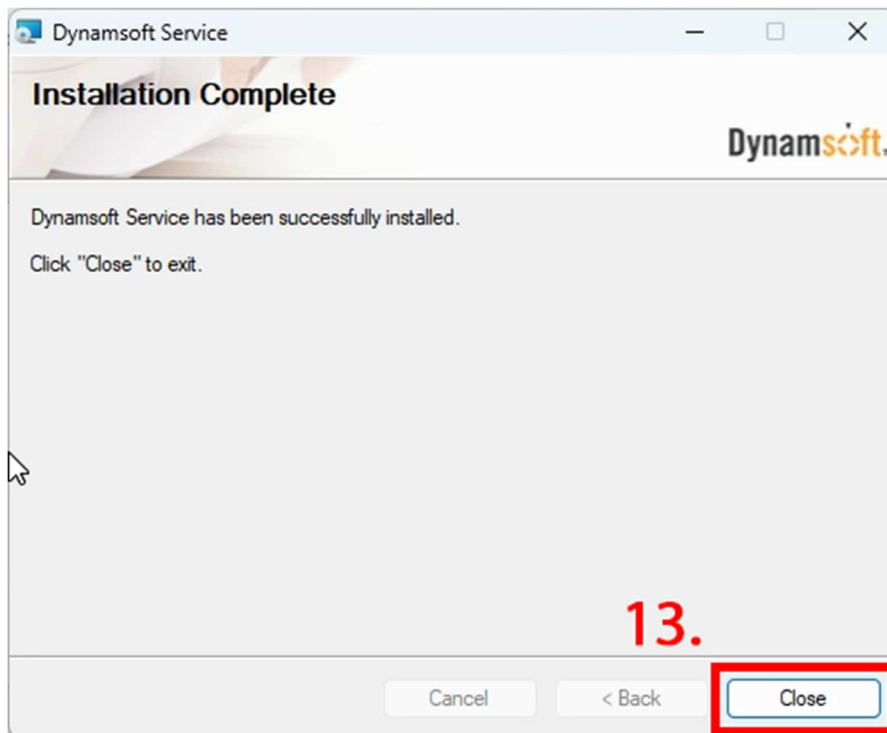


14. The web browser may need to be refreshed if this screen is displayed:

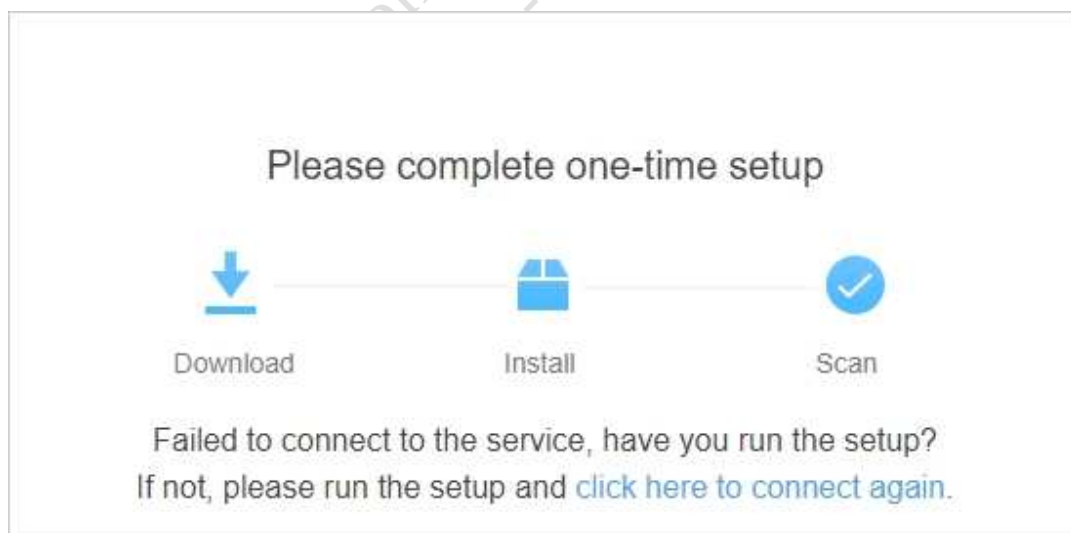


15. Select the preferred TWAIN device from a list in the top right corner (“Intraoral USB Sensor” for IOX with EzSensor HD and “VCaptureSW” for 2D / 3D EOX devices).

16. Click “Scan” to arm the device:

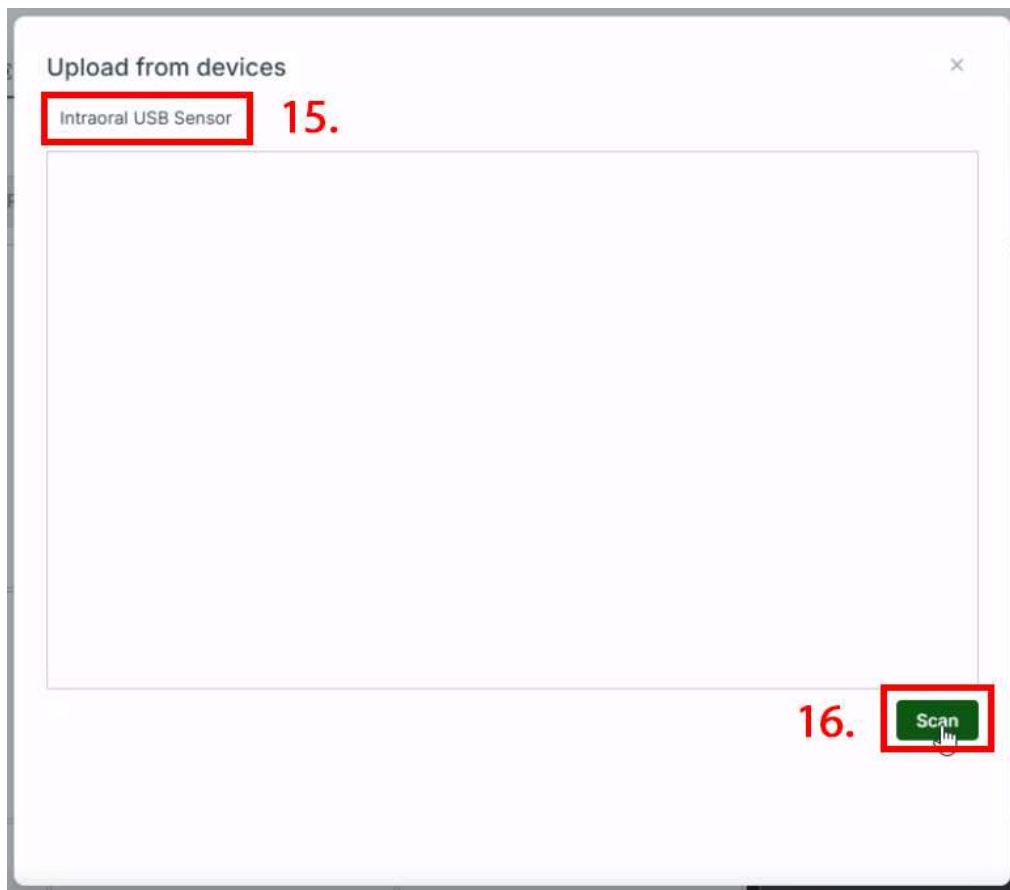

17. Proceed with acquiring images. (The window should close automatically after capturing a single extraoral image.)

18. When finished taking a series of intraoral images, click on the EzSensor UI and press the “ESC” (escape) key on the keyboard or right-click on the dark blue circle with the pink “V” icon on the Windows Taskbar and choose “Close window” to complete the series:

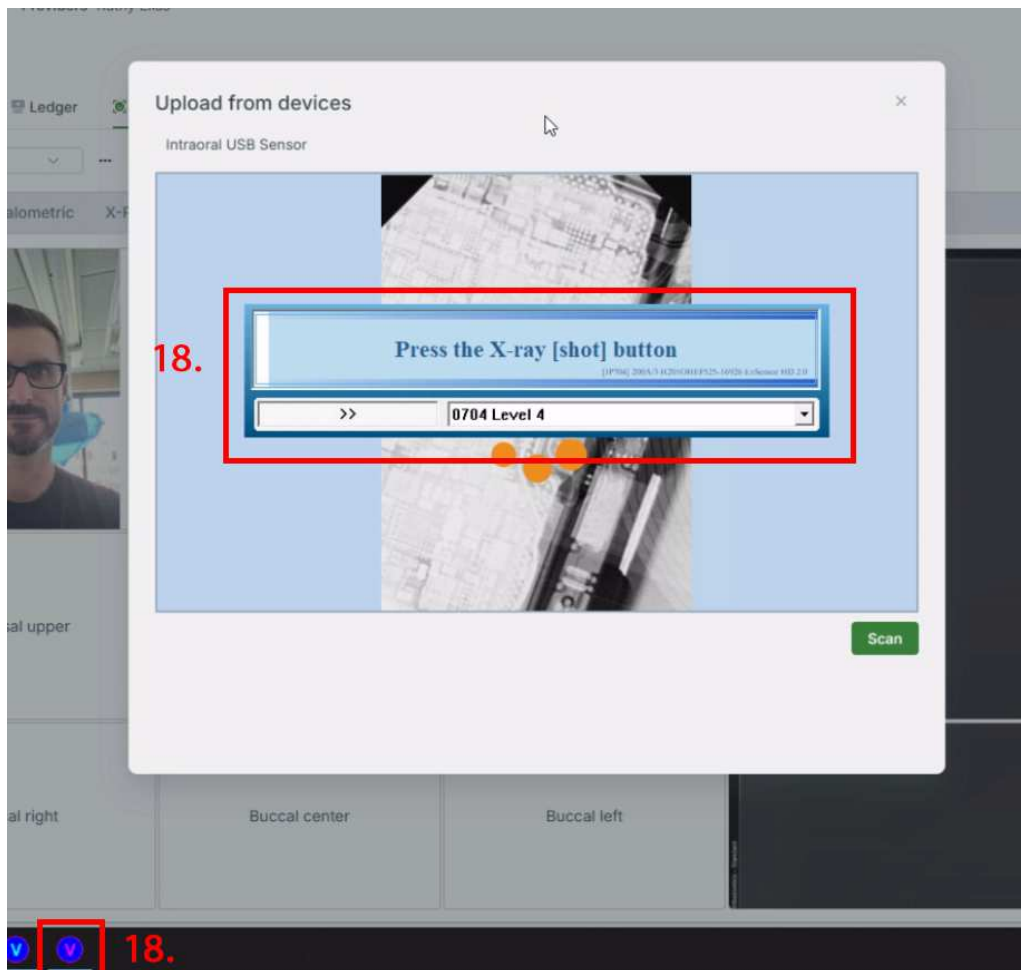

19. Finally, all captured images will be visible in the lower right corner of the patient's record where the user can then drag each one into the appropriate mounting square within the layout:

