


Exposure Switch Configuration Options for Extraoral Installations

1. Exposure Switch (Handheld remote)
2. Exposure Switch (Doorbell switch)

Setup for Exposure Switch (Handheld remote)

- Run the exposure switch (handheld remote) cable from the exposure location to the Vatech extraoral unit and connect the cable to the appropriate connection at the base of the unit.
- Mount the exposure switch (handheld remote) holder.
- Place the exposure switch (handheld remote) into the holder.

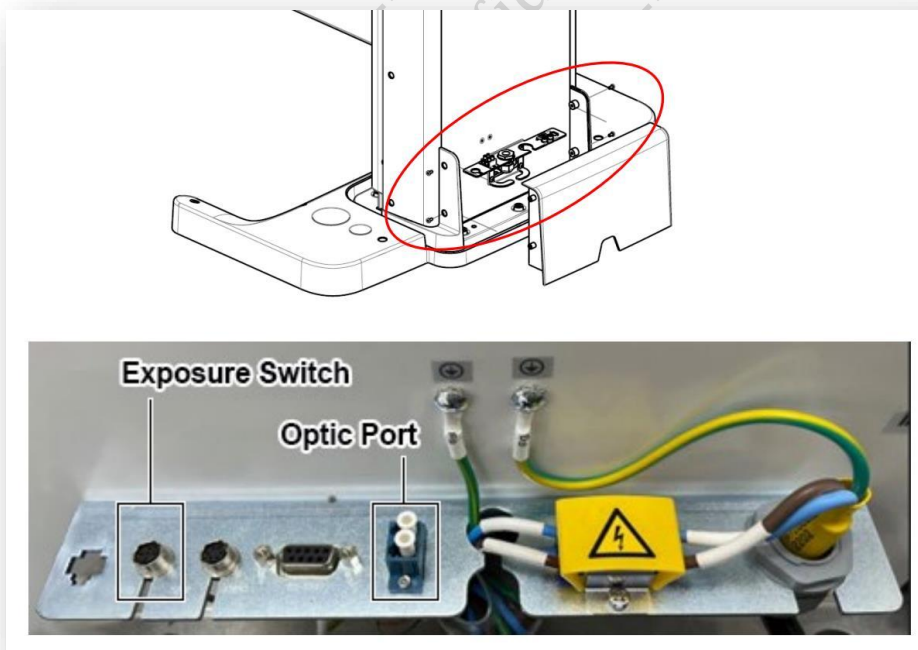

Setup for the exposure Switch (Doorbell switch)

- Run the exposure switch (doorbell switch) cable from the exposure location to the Vatech extraoral unit and connect the cable to the appropriate connection at the base of the unit.
- Appropriately wire the exposure switch cable to the doorbell switch plate. (See diagram below)
- Mount the doorbell switch plate to the wall.



Confidential