

**Vatech America, Inc.**  
500 Frank W Burr Blvd, #710  
Teaneck, NJ 07666  
Tel) 201-210-5028  
Fax) 201-601-0859



**EzSeries: EzDent-i to CleverOne Conversion:**

**IT team should complete updates for EzServer prior to NEW installation or Upgrade.**

I.T. should see the following instructions on updating to CleverOne on FAQ guidance

**Step 1: Ensure proper backups before starting.**

<https://vatechamerica.com/faq/software/how-to-backup-ezdent-i-software>

**Step 2:** Download the following files before proceeding:

A. DOWNLOAD CLEVER ONE:

<https://vatechamerica.files.com/f/357917281b8138f1/Clever%20One>

B. DOWNLOAD IMPLANT LIBRARY:

<https://vatechamerica.files.com/f/21c069ec449c4404/Implant%20Data>

**Step 3:** Follow the CleverOne Guide: Updating EzServer to version 6.2.3

<https://vatechamerica.com/faq/software/CleverOne>

**Step 4:** If the database needs to be moved to a different PC/Server follow the migration guide:

Migration Guide for EzServer Series 6 and higher: Please note that EzServer 6.2.3, Consult Data and Implant Data will need to be installed on the New Server. Old server should be updated to at least 5.5.3 or newer but it is recommended to update to EzServer 6.2.3 with backup each update step.

<https://vatechamerica.com/faq/vatech-2d-ims/how-to-move-or-merge-ezdent-i-database-6-2>

**Step 5: Workstation Installation:**

Follow the CleverOne installation FAQ guide to install CleverOne only\* on Workstations.

<https://vatechamerica.com/faq/clever-one/CleverOne>